

Intravenous Cannula Insertion Kit

Instructions for Use

Intended Use: The intravenous cannula insertion kit is used to aid in the procedure of inserting a peripheral intravenous cannula (PIVC) by trained medical staff.

Read all manufacturers "Instructions for Use" carefully before use

Setting Up Instructions: Peel off the tracking label, fill out the required details and attach it to the patient records. The intravenous cannulation kit contains a non-sterile outer pack and a sterile inner pack.

1. Open the insertion kit's outer pouch as per your hospitals guidelines
2. Remove all contents and set up as per your hospitals guidelines.
NOTE: Only contents of insertion kit's inner pack are sterile, the packaging of all additional medical devices are NON-STERILE.
3. Open the insertion kit's sterile inner pack using aseptic technique as per your hospitals guidelines.
4. Open additional devices as per manufacturer's instructions and your hospital's guidelines.

Contents - Outer Pack:

The outer pack is non-sterile and contains individually packaged medical devices.

Contents - Inner Pack:

The contents of the inner pack are sterile.

STERILE	EO
---------	----

Cautions:

1. Do not use if package or contents are damaged.
2. Store at controlled temperatures (15°- 25°C). Excursions permitted to 30°C.
3. Do not re-use / re-sterilise products. ②
4. For single use only.

For more information contact your ITL Australia Account Manager

Manufacturer

ITL Healthcare Pty Ltd
Unit 1, 63 Wells Road
Chelsea Heights, VIC 3196
Australia.

Distributed By

ITL Australia Pty Ltd
[h] 1300 188 493
[t] +(61) 3 8773 3006
[f] +(61) 3 8610 0300
[e] mysales@itlaustralia.com
[w] www.itlaustralia.com

